



Strong like rock

G-ænial®
Universal
Injectable
from GC

High-strength
restorative composite

GC

G-ænial® Universal Injectable from GC


Strong like rock

For all cavity classes No covering layer required

Dr Ahmed Al-Jaff, Iraq



252% Al radiopacity
for perfect follow-up

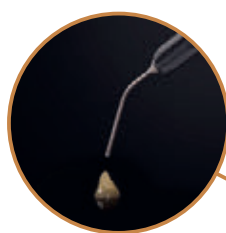

Perfectly designed: no
uncontrolled material flow-out and
easy separation from the tip




Bendable tips
for easy access
to any cavity



No slumping:
everything stays
where you place it



Easy extrusion
for maximum comfort

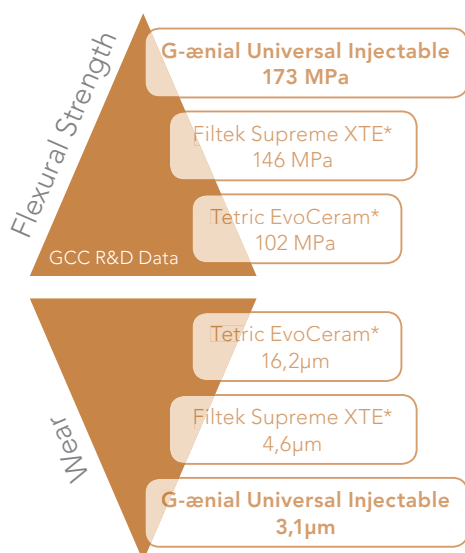


Ultra-fine **Barium**
fillers (150 nm) &
high filler load

**Full-coverage
Silane Coating (FSC)**
Technology
improving the
**dispersion &
adhesion** of the
fillers in the matrix

Exceptional properties
No stickiness
Low extrusion pressure
High radiopacity

* Not trademarks of GC

Start injecting!

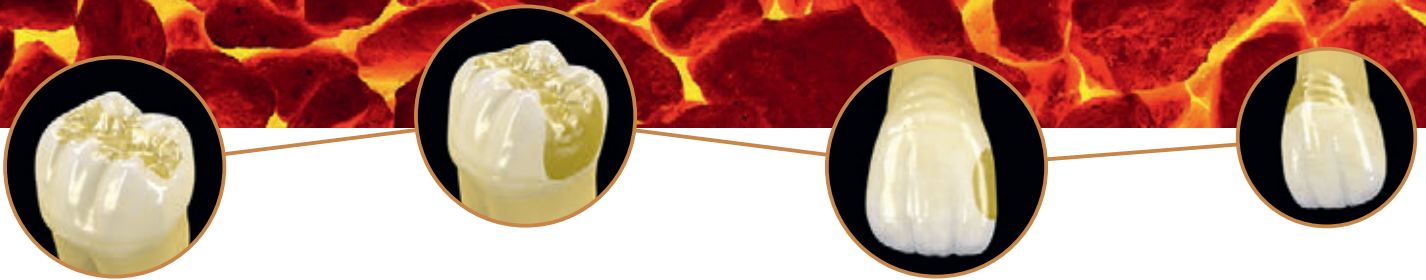

Class II restoration



Initial situation showing interproximal caries



Removal of the old restoration & caries removal



Undercuts can easily be reached thanks to the bendable tips



Building the proximal wall using the enamel shade JE



Removal of the sectional matrix after building the wall



A Class I procedure can now be followed; the dentin layer is created with the A4 shade



Placement of the final enamel layer with the JE shade



Final situation after finishing and polishing the margins

Dr Javier Tapia Guadix, Spain

Class III restorations



Initial Class III cavities



Injecting G-ænial Universal Injectable (A2)



Immediate post-operative result



Recall situation showing perfect integration

Dr Ahmed Al-Jaff, Iraq

G-ænial® Universal Injectable from GC

Enjoy superb aesthetics & long-lasting gloss!



Dr Florian Klumpp, Germany



Dr Ali Salehi, France

	Standard shades											Inside shades			Outside shades	
	XBW	BW	A1	A2	A3	A3.5	A4	B1	B2	CV	CVD	AO1	AO2	AO3	JE	AE
Syringe 1mL (1,7g)	901471	901472	901473	901474	901475	901476	901477	901478	901479	901480	901481	901482	901483	901484	901485	901486



901503 - Dispensing Tip
Long Needle (x30)



Also discover GC's universal bonding G-Premio BOND:

- 009030 – G-Premio BOND Kit 5mL
- 009036 – G-Premio BOND Bottle Refill 5mL
- 009032 – G-Premio BOND 3-Bottle Refill, 3x5mL
- 009034 – G-Premio BOND unitdoses, 50 pieces

GC EUROPE N.V.
Head Office
Researchpark
Haasrode-Leuven 1240
Interleuvenlaan 33
B-3001 Leuven
Tel. +32.16.74.10.00
Fax. +32.16.40.48.32
info.gce@gc.dental
http://www.gceurope.com

GC UNITED KINGDOM Ltd.
Coopers Court
Newport Pagnell
UK-Bucks. MK16 8JS
Tel. +44.1908.218.999
Fax. +44.1908.218.900
info.uk@gc.dental
http://uk.gceurope.com

